

C-Smile T1


Medical Grade Transparent Face Mask

C-Smile T1 meets the NHS Transparent Face Mask Specification | Relevant clauses of BS EN14683:2019 & BS EN166:2002

INSTRUCTIONS FOR USE

INTENDED USE

The C-Smile T1 transparent face mask is a non-sterile medical device and recommended for single shift use. Its primary purpose is for healthcare professionals, healthcare workers, first responders and other personnel to protect the patient from infection via direct breath, vapour, liquid splashes, and droplets at the same time as letting their facial expressions being seen.

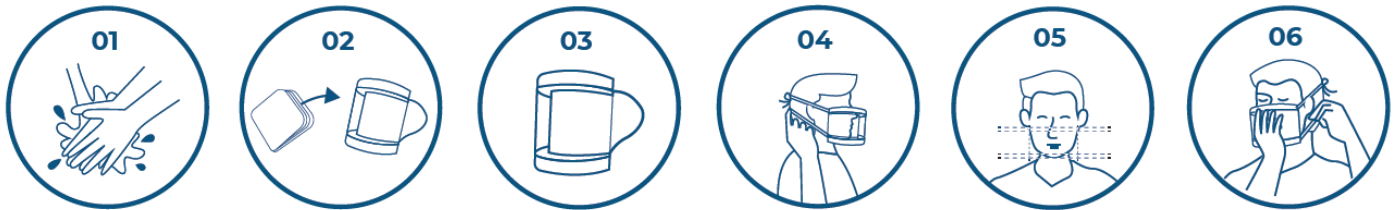

DONNING THE MASK

1. Wash hands thoroughly for 20 seconds before touching the mask.
2. Carefully remove the T1 from packaging.
3. Ensuring the mask is the correct way up – Thinner foam at the top, thicker foam at the bottom, pull the T1 over face and position strap around the back of head.
4. Tighten to the desired length by tying a knot in the elastic behind the head. Make sure the mask is comfortable but secure during movement.
5. Line up the nose contour two thirds of the way down the bridge of the nose*.
6. Ensure the nose foam is central and firmly press to mould the foam to the contours of face.
7. Do not touch mask again unless you plan to remove it.

*The T1 should sit comfortably below your glasses, if you experience any interference lower the mask on the bridge of the nose and ensure it is central.

REMOVING THE MASK

1. Remove mask by pulling elastic over your head not touching the surface of the mask.
2. Dispose of the mask in a covered waste bin immediately after removing from your face.
3. Wash your hands thoroughly after disposing of your mask.

DESCRIPTION	MATERIAL
Transparent Plastic Shield	Anti Fog PET
Comfort Nose Band	Pu Foam
Comfort Chin Filter Frame	Pu Foam
Filtration	Meltblown + Spunbond PP
Strap	Latex Free Elastic



CAUTIONS

1. This is a disposable, single shift use product, do not use if packaging is damaged.
2. Do not share product to prevent cross infection.
3. Do not use the mask if it damaged e.g. ripped or torn.
4. Do not try and clean the mask it is not a reusable product, it should be disposed of safely after each shift use.

REGULATORY

Meets requirements of the MDR for medical devices 2017/745.
Meets the splash resistance performance requirements for type IIR surgical masks (EN14683:2019+AC:2019, chapter 5.2.7).
Meets the microbial cleanliness (bioburden) performance requirements for type IIR surgical masks (EN14683:2019+AC:2019, chapter 5.2.7).
Meets the breathability performance requirements for type IIR surgical masks (EN14683:2019+AC:2019, chapter 5.2.7).
Meets the bacterial filtration efficiency performance requirements for type IIR surgical masks (EN14683:2019+AC:2019, chapter 5.2.7). Meets the EN166:2002 resistance to fogging (anti-fog) performance requirements.



EMERGO EUROPE
Prinsesgracht 20
2514 AP The Hague
The Netherlands



PolarSeal Tapes & Conversions Ltd
Guildford Road Trading Estate
Farnham
Surrey
GU9 9PZ
United Kingdom

W: www.polarseal.net/ectovise
T: +44 1252 726000